

CASE EXAMPLES--BEFORE & PROGRESS PICTURES

CASE 1



1 MONTH 15 DAYS



CASE 2



1 MONTH 4 DAYS



CASE 3



1 MONTH 15 DAYS



CASE 4



3 MONTHS 6 DAYS




THIS WAS DONE WITHOUT STRIPPING ANTERIOR TEETH

EXAMPLE OF HAWLEY RETAINER WITH VECTOR III® Duo-Tee® Springs



TOP VIEW

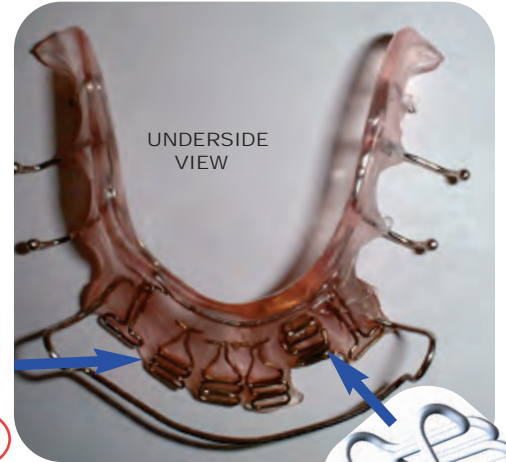

UNDERSIDE VIEW



THE AREA INSIDE THE RED CIRCLE IS THE PORTION EMBEDDED IN ACRYLIC

torquing type

EXAMPLE OF NEW WORLD RECORD VECTOR III® Duo-Tee® Springs ON HAWLEY BITE-JUMP

START



PROGRESS AT 4 DAYS!



A COST EFFECTIVE ALTERNATIVE TO INVISIBLE ALIGNERS

TREATMENT INDICATIONS OR USES OF

VECTOR III® Duo-tee® Springs:

U. S. Patent #6,604,943

I. Mixed dentition, with "baby teeth" age 6-12 years:

1. **Cross-bites** (x-bites)
 - a. Adult front teeth (anterior).
 - b. Primary teeth (baby teeth) arch width in conjunction with expanding type removable appliances.
2. **Alignment & rotation**, tipping front (anterior) permanent teeth (upper & lower).

II. Adult dentition, (age 12 + years) "adult teeth":

1. **As a primary treatment** with removable appliances (no pre-treatment with fixed appliances):
Limited to indicated types of malocclusions as per doctor. Note functions I-VI, thus a significantly higher percentage of patients can be treated without fixed appliances (braces) than prior art or current treatment choices.

Functions indicated: I thru VI are:

- | | |
|---------------------------------|--|
| I Rotate teeth | IV Combination: tip, translate, rotate, intrude, extrude |
| II Tip teeth labially/lingually | V Torque roots labially/lingually |
| III Translate (bodily move) | VI Intrude/extrude teeth |

2. **An early finish** or brace removal option.

When detailing and minor rotations on upper & lower teeth are not completed, they can be finished with **VECTOR III® Duo-Tee® Springs** added to the Hawley retainers.

3. **In retention** following active treatment with fixed appliances.
 - a. Correction of relapse
 - b. Preventative, added to 1st retainers when the pre-treatment position of the teeth indicate a higher possibility of relapse.

III. **As a dental adjunct to esthetic dentistry**, for minor anterior tooth alignment before bridges, veneers,

These upgraded retainers can continue to be used for retention.

**CHECK WITH YOUR DOCTOR TODAY TO SEE IF
VECTOR III® DUO-TEE® SPRINGS ARE RIGHT FOR YOU!**

©All Rights Reserved **VECTOR III®** 2001-2015

see our website at: www.VECTORIIIORTHO.com

U. S. Patent # 6,604,943

Form #103

PLEASE TAKE ONE

IMPORTANT PATIENT INFORMATION

VECTOR III® Duo-Tee® Springs

U. S. Patent # 6,604,943

THE NEWEST WAY TO
STRAIGHTEN TEETH

WITHOUT BRACES!!

FAST

EASY

SAFE

PAINLESS

COST EFFECTIVE FOR
PATIENTS AND DOCTORS

ABILITY TO MOVE ROOTS AND TEETH

BIOANALYTIC™ MECHANICS +
VECTORLOY™ WIRE =
REVOLUTIONARY RESULTS

Patent # 6604943 **VECTOR III®**

© ALL RIGHTS RESERVED 2001-2015